Wartungsset L2-D2 Serie Art. Nr. 211390 / Service Kit L2-D2 Series Art. 211390 Kit de maintenance Série L2-D2 Réf. 211390

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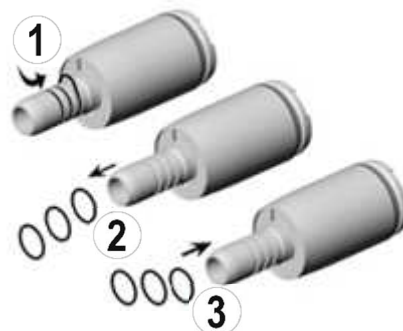
[Squeeze the O-ring together between thumb and forefinger to form a loop.](#)

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[Slide on the new O-rings with a pair of tweezers.](#)

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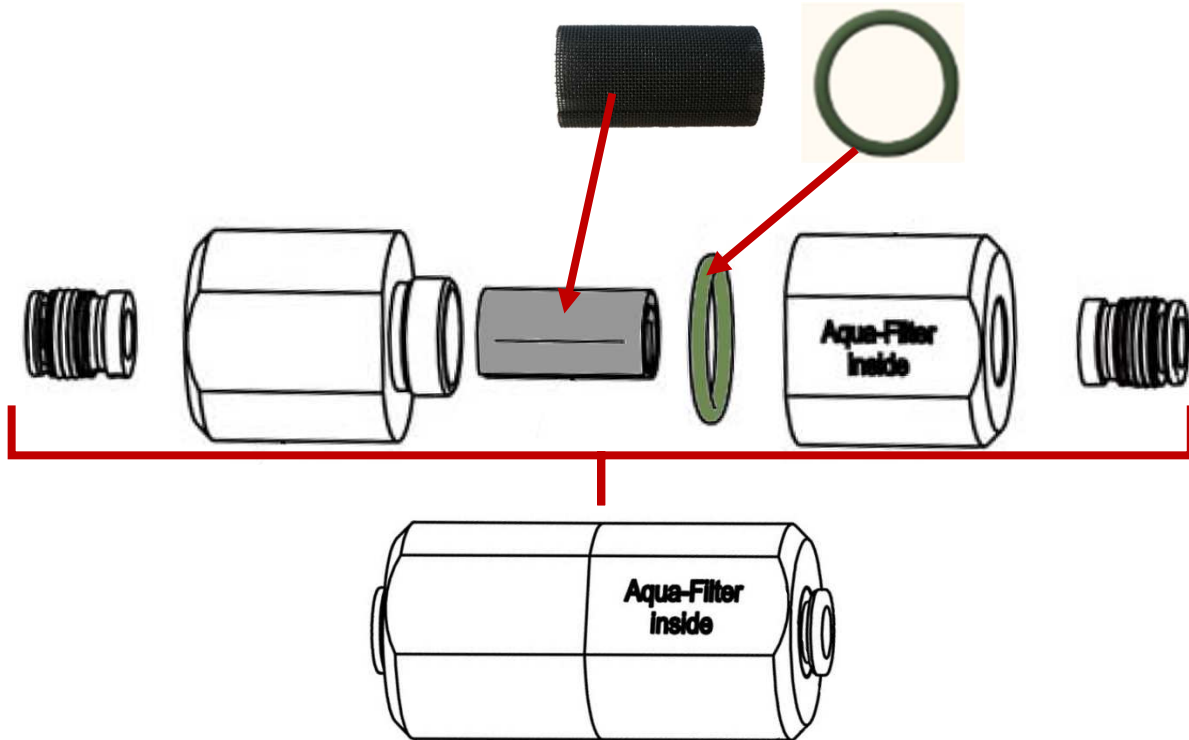
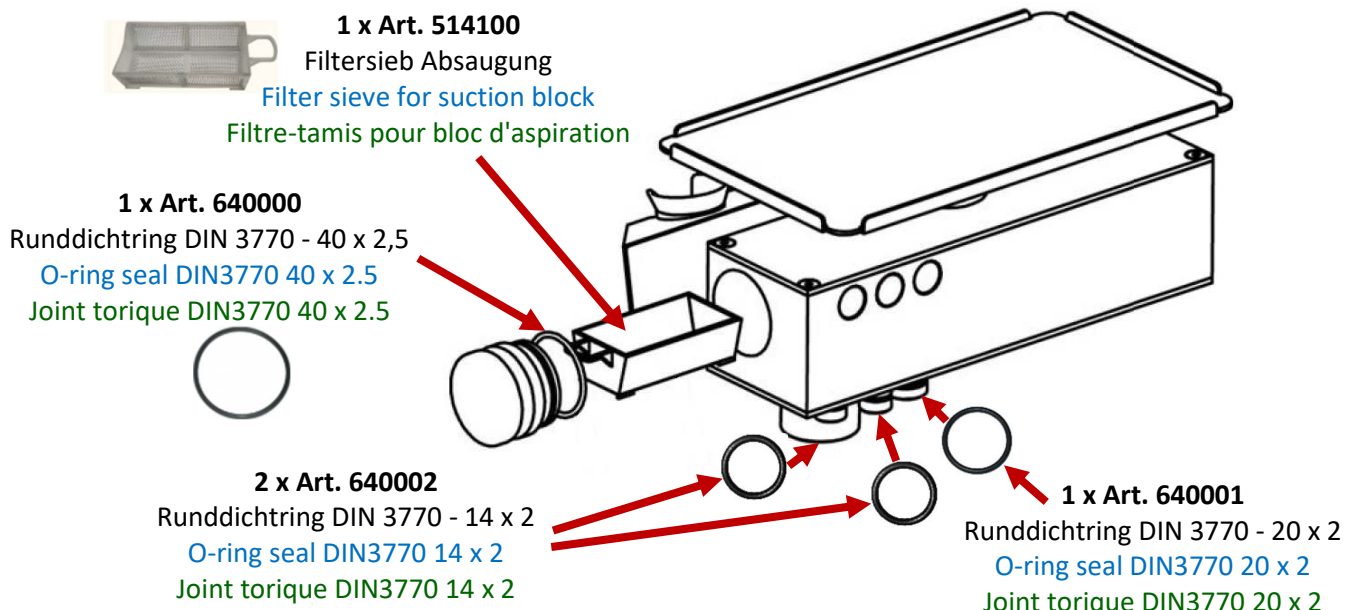
DKL CHAIRS L2-D2 Series Water-Air Inlet Blocks until 01.02.2024, up to serial number 2444, excluding units converted to the WAS-Unit (Water-Air Inlet).

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**Filter element Air inlet block
with O-ring seal**

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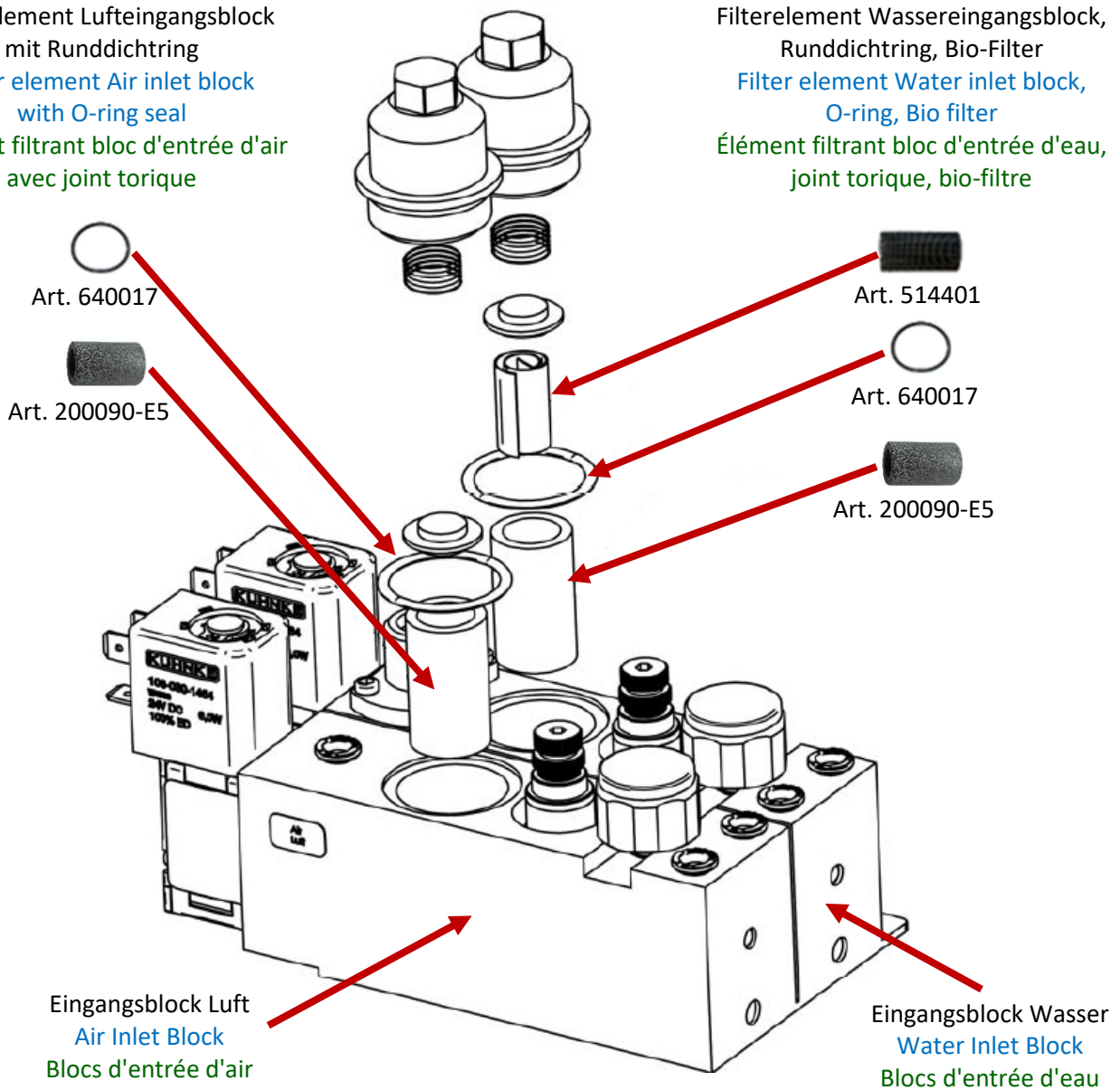
**Filter element Water inlet block,
O-ring, Bio filter**

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Switch off the unit before changing a filter!

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CAUTION! The supply and shut-off of water and air pressure is controlled by the air filter! Therefore, always set the air filter to OFF first (pressure release for air and water) before removing either of the two filters. Only set the air filter to ON when the water filter is also inserted and is set to OFF or ON! Never set the water filter to "FILTER" if the air filter is not set to OFF or FILTER!

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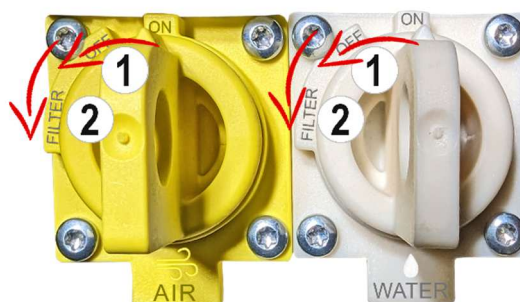
Turn the filter caps from ON to OFF. The filter caps will lift slightly for pressure release.

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Press the filter caps down and continue turning to FILTER and pull out the filter caps with the attached filters.

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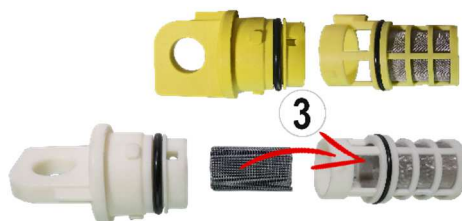
Pull the filters off the filter caps and replace the filter. For the water filter, replace the Bio filter along with the filter.

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
Reinsert the filter caps with the new filter attached and turn to ON. **CAUTION!** Always turn the water filter to ON first!

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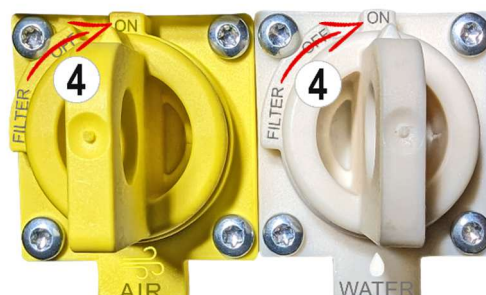
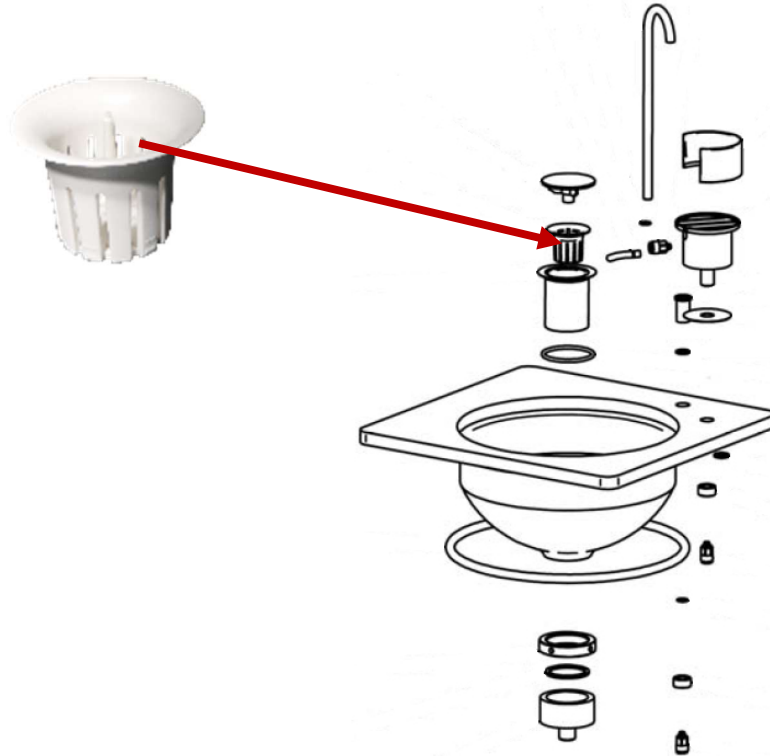
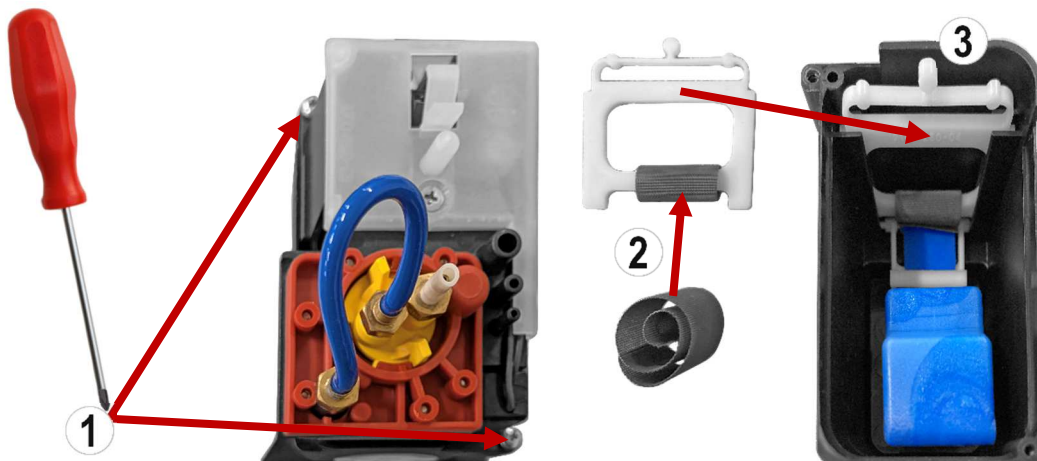
Replacement of Valve diaphragm for suction air shut-off valve DÜRR

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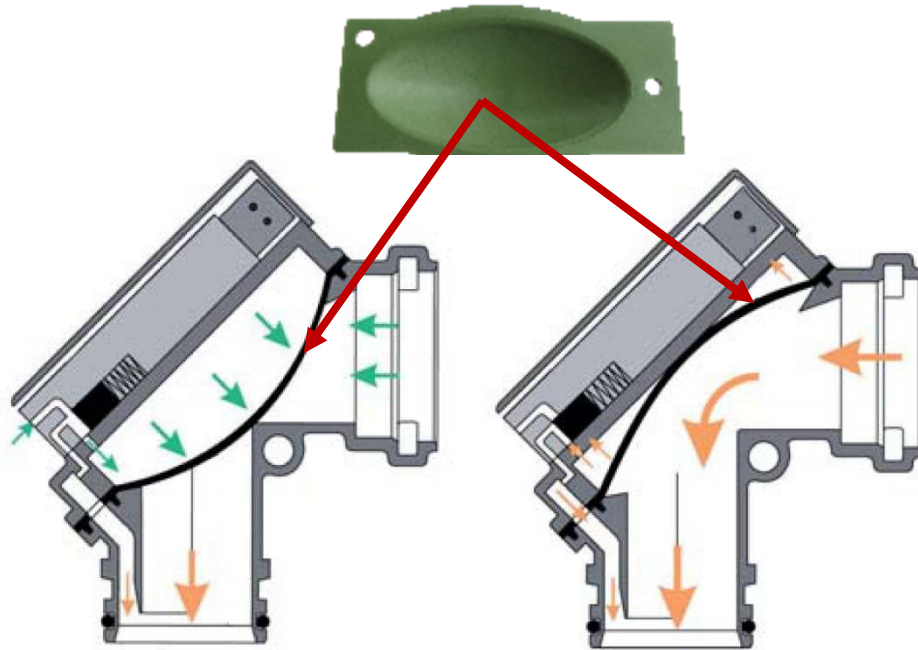Perform maintenance work for the amalgam separator "CAS 1 Combi-Separator", the " CS 1 Combi-Sepamatic 24 V AC" or the "Spittoon valve 3" according to the maintenance instructions of the company DÜRR!

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DKL CHAIRS GmbH • Tel. +49 (0)551 5006-0 • info@dkl.de • www.dkl.de