

Replacement of the L1-D1 Instrument Holder Large and small suction with light barrier

Art. No. **210476GS** instrument holder for large suction and **210477KS** for small suction.

Disassembly of the Instrument Holder



Preparation: Switch off the unit.

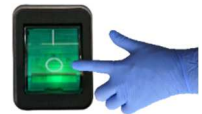


Figure 1: In the center of the inner radius of the instrument holder, there is a fixing pin (diameter 2 mm). Push this pin through to the back into the strip of the instrument holder support. Use a sturdy paper clip or a needle with a maximum diameter of 2 mm. A new fixing pin is included with the replacement instrument holder.



Figure 2: There are two holes below the instrument holder. Insert angled internal circlip pliers into these holes and squeeze the pliers. This releases the tension of the locking mechanism, and the instrument holder can be removed upwards.



Figure 3: Carefully pull the instrument holder upwards. It is essential to ensure that the cable connections are not damaged.

Figure 4: Disconnect the plug connection of the instrument holder.

Assembly of the Instrument Holder

- Assembly of the instrument holder is carried out in the reverse order of disassembly.
- **ATTENTION!** When inserting the instrument holder, make sure that the cable connections are not crushed or damaged! Insert the instrument holder until it stops.
- Switch the unit back on after assembly and carry out a functional test of the instrument holder (light barrier).