Kunstleder-Kollektion

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Imitation leather - Collection

- smooth (and with some versions shiny)
- surface with a skin-friendly coating
- waterproof and can be disinfected
- biocompatible according to EN ISO 10993

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Properties:

This material has a smooth (and withsomver-sions shiny) surface with a skin-friendly coating. It is waterproof and can be disinfected. Imitation leather from the DKL-collection complies with the relevant regulations of the EC directive 93/42/EWG on medical devices. DKL imitation leather is biocompatible according to EN ISO 10993.

Care Instructions:

Never use disinfectants containing alcohol. We recommend applying proPad – ready-to-use wipes – for disinfecting alcohol-sensitive surfaces, in particular the upholstery of treatment chairs. Clean the upholstery with DentaClean and a soft sponge at the end of each treatment day. Never use cleaning agents or household cleaners! After being cleaned, the upholstery should be wiped dry with a cloth. Then apply DentaProtect (wipe dispenser), our sealant for imitation leather.

Soiling / Discolouration

Light colours such as white, yellow, light green, light blue, beige or orange, are particularly susceptible to soiling or discolouration. Residual humidity from disinfectants enhances the migration of colourants from textiles. Particularly aggressive discolouration stains are caused by belts and suede leather clothes. Carefully wipe the upholstery dry after the residence time of the surface disinfectant! Soilings can be removed with water and a sponge or a brush. Please note: discolourations caused by jeans or other textiles excluded from manufacturer's guarantee. You will find detailed information on this topic in the DKL care brochure, which is available for download on our website www.neuepolster.de. Please note that, for technical reasons, the colours may differ slightly from the original surfaces.

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PLEASE NOTE: We do not accept any liability for discolourations caused by textiles on white, light-coloured, yellow, light green, light blue, beige and orange upholstery parts.

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DKL GmbH